

# Sending Autorefractor and Lensometer Data to Topcon CV-5000S

Version 6.0.29 +

\*THE FOLLOWING FIELDS MUST EXIST IN THE MEDICAL RECORD TEMPLATE

## Autorefractor:

Sphere OD & OS

Cylinder OD & OS

Axis OD & OS

Distance PD

Near PD

## Lensometer:

Sphere OD & OS

Cylinder OD & OS

Axis OD & OS

Add OD & OS

Add 2 OD & OS

H Prism OD & OS

V Prism OD & OS

## Creating A Button to Send Values

1. In records, navigate to HER Settings then choose Edit Medical Records Templates.
2. In the Edit Records Window, click on Add New Field at the bottom of the window.
3. Change the Text to "Send to Topcon"
4. Make the Type Button
5. Choose Command > Pre-Loaded Application > Export Topcon LM/AR
6. Fields to Send to Machine:
7. **Input\_File\_Location:** This line MUST BE EXACT – You should copy/paste this to make sure it is correct.
8. For the rest of the values below, double click on the value and it will ask you for the field ID. The field ID is the value in (parenthesis)
9. One done **CLICK SAVE!**
10. Right click template – Add Field to Tab
11. Stop editing and save changes.